



BIG Bone Injection Gun

Leading Intraosseous Devices

A NEW SUPPLEMENT TO THE B.I.G. PRODUCT LINE IS READY!

ONE BAG TWO BIGS

WaisMed is proud to offer your clients a useful addition to the B.I.G. product line - a protective bag that can contain both B.I.G devices for treating any patient at any age.

The orange bag has colourful clear instruction cards to assist the user with finding the exact location and depth calibration for insertion of the B.I.G (4 Adult locations).

The orange bag provides the BIG with additional protection for its 'time on the field' - whether it is in the ambulance, the helicopter or inside a personal medic bag.



Pediatric B.I.G - RED

Recommended Penetration Depth in Proximal Tibia

0-3 Years	0.5-0.7 cm
3-6 Years	1-1.5 cm
6-12 Years	1.5 cm

Attention - Please refer to instruction for use

www.waismed.com mail@waismed.com

WWW.WAISMED.COM

ONE BAG TWO BIGS

WaisMed Ltd.
Leading Intraosseous Devices

Adult B.I.G - BLUE

Recommended Penetration Depth

Proximal Tibia/Humerus	2.5 cm
Medial Malleolus (Distal Tibia)	2 cm
Distal Radius	1.5 cm

Attention - Please refer to instruction for use

www.waismed.com mail@waismed.com